

From the manufacturer



Universal Tagger

Applies the full line of ALLFLEX® tags giving you ultimate flexibility in how you identify your cattle.

Durable


Constructed with high quality diecast aluminum with replaceable plastic inserts that will last.

Comfortable

Deeper jaws and comfortable grip make lining up the applicator and tagging cattle a breeze.




Applying Visual Tags

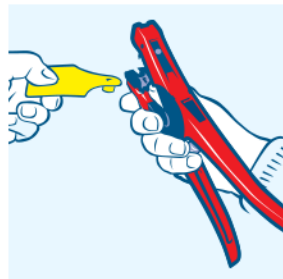


Step 1: Place female tag

Place the visual tag under the clip by depressing the clip. It is recommended that adult cattle are restrained in a head bail for tagging.

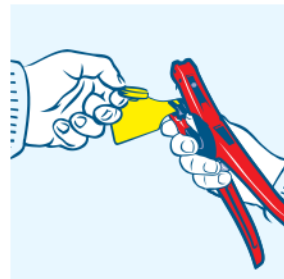


Step 2: Place male tag

Slip male tag completely onto the application pin.

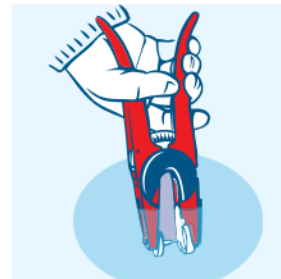

Step 3: Sanitize

Dip jaws of applicator holding tag into antiseptic or disinfectant solution. Use disinfectant at recommended dilution ratios. Failure to do so may cause irritation.

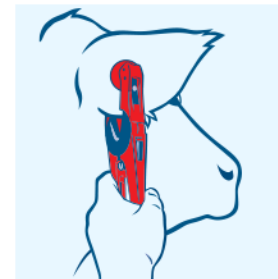

Step 4: Apply

With the correct position located, apply the tag firmly, always with the male tag entering from the back (outside) of the ear.

