

From the manufacturer



Premium applicator. Re-engineered for modern tagging systems and farm labor today.



**Smooth Application**

The UTT3S ergonomic grip requires low force, resulting in less hand fatigue and easier application of multiple tags.



**Simple Design**

Requires much less force to squeeze when applying tags, for less hand fatigue and easier application of tags in multiple animals.




**Safe Solution**

Designed with reduced pinch points for the user, the UTT3S is also quick and quiet, reducing animal startle reflex.




**Applying Visual Tags**

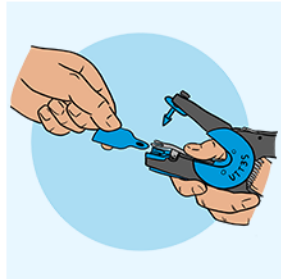


**Step 1: Place female tag**

Place the visual tag under the clip by depressing the clip. It is recommended that adult cattle are restrained in a head bail for tagging.

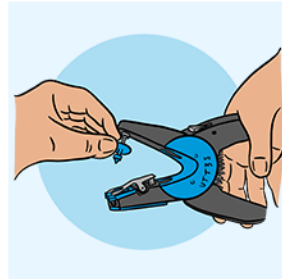


**Step 2: Place male tag**

Slip male tag completely onto the application pin.

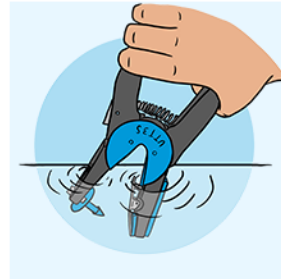

**Step 3: Sanitize**

Dip jaws of applicator holding tag into antiseptic or disinfectant solution. Use disinfectant at recommended dilution ratios. Failure to do so may cause irritation.

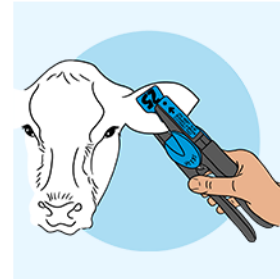

**Step 4: Apply**

With the correct position located, apply the tag firmly, always with the male tag entering from the back (outside) of the ear.

